





Healthcare

Elmasteam 8 med

Powerful cleaning in the CSSD

- Long steam availability with 8 bar steam pressure
- Individually adjustable pressure ranges "soft" and "strong"
- Automatic "Eco-mode" saves costs and energy
- Run-dry-save heating and intelligent decalcification cycles
- Luer-Lock-connection for perfectly cleaning of hollow body instruments


Elmasteam 8 med Removes easily and gently even tenacious soiling


Certain instruments, such as burrs, drills and electro-surgical instruments, often do not become sufficiently clean in the standard washing and disinfecting unit.

The Elmasteam 8 med steam jet cleaner is the ideal tool to preclean such instruments and smoothly removes even tenacious contaminations like burnt-in tissue on electro-surgical forceps. So the valuable instruments are preserved for longer time.

The insides of hollow body instruments can be connected directly to the Luer-Lock-connection at the handpiece or can be cleaned with the optionally available adapter set.



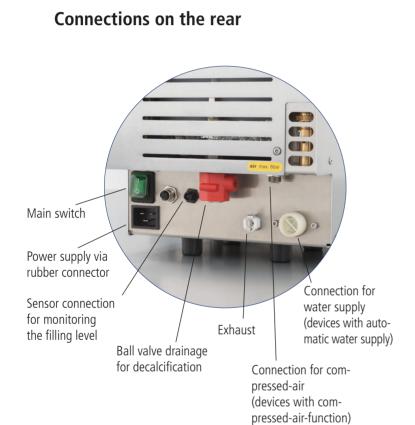

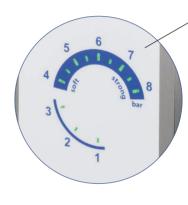

Digital manometer Display of desired and actual steam pressure

Multi-function switch

- "Strong" cleaning mode with 6-8 bar steam pressure
- "Soft" cleaning mode with 4-6 bar steam pressure
- Adjustment of water hardness and chlorine content for decalcification in time as well as adjustment of the time until the steamer switches to Eco-mode



Application examples before and after cleaning

Hemostatic forceps



Removal of blood






Removal of blood

Diamond burr



Removal of bone meal

Diamond burr - Detail

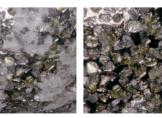

Anwendungsbilder ©www.mittelohr.de



Elmasteam 8 med Great performance – tailor-cut technology

The high performance of the Elmasteam 8 med is due to the optimised piping system with larger cross sections, the intelligent pump control, the increased heating power (60% more) as well as the optimised heating position.

The intelligent control system of the Elmasteam 8 med calculates, by means of preset water hardness and the intensity and frequency of use, the proper time for decalcification. The user is led step by step through the decalcification process.

Due to this integrated decalcification programme and the protection against dry running of the heating, the Elmasteam 8 med is highly reliable and fit for use for a long time. By taking these measures and provided that the unit is being operated properly and in one-shift operation, the usual warranty can be extended to 3 years.



Elmasteam 8 med

With thought-out accessories for perfect cleaning results



Technical Data

Mains voltage (V)	220 - 240	Pressure tank volume (litres)	5.0
Mains frequency (Hz)	50 / 60	Max. filling volume (litres)	3.8
Max. power consumption (W)	3120	Dimensions W/D/H (mm)	285 / 350 / 520
Steam temperature in the tank (°C)	185	Weight empty (kg)	20.0
Steam temperature at the nozzle (°C)	160	Registered as medical device	class 1
Operating pressure (bar)	8.0		

The scope of delivery includes:



 $\begin{array}{l} \mbox{Particle filter} \cdot \mbox{Tweezers} \cdot \mbox{Maintenance} \\ \mbox{hose} \cdot \mbox{Rinsing set} \cdot \mbox{Extra tank cap} \cdot \\ \mbox{Universal top for handpiece} \end{array}$

Further accessories and optional unit functions:





Adapter set consisting of 8 parts incl. pliers for changing the adapters



Compressed-air (optional function)



Canister with level indicator

*3 years warranty for proper use in accordance with the operating manual. Wear parts are not covered by the warranty.

